

P-BAG – URINE BAG FOR MEN AS WELL AS WOMEN

P-BAG is the perfect replacement for bedpan and urine bottle in hospitals, old age homes, nursing homes as well as in homecare.

P-BAG is a urine container with an anatomically formed funnel that allows not only men but also women to use the urine container. P-BAG is very easy to use.

P-BAG consists of the reusable (by the same person) funnel with a smooth surface that minimises the fluid that stays in the funnel and a disposable polyethylene (PE) bag that can hold up to 1.5 litres.

The bag is either blue (old age homes / homecare) or white with indicator that allows urine volume assessment in 100 ml steps (hospital / nursing homes). The P-BAG can also be laid on a scale for more accurate volume calculations. A P-BAG weighs 50g.

The bag has a one-way valve that stops the outflow of urine as well as the spreading of unpleasant odour. Therefore it is not necessary, as in the case of bedpan and urine bottle, to empty and clear the urine container immediately after use. P-BAG is only emptied when full or after 24 hours of use.



What bedpan and urine bottle has in common

For a very long time, old age people, disabled people or people unable to leave the bed has been obliged to use different kind of urine collection containers. Today you give women a bedpan and men a urine bottle.

Both systems are:

- open containers with a high possibility that urine is spilled during handling of the container.
- uncomfortable for the patient and due to their open construction, spreads an odour as soon as they are used.
- necessary to empty and clean directly after use. Many different patients are using the same container.

Where can the P-BAG be used

- In hospitals,
- In old age and nursing homes,
- during transport of patients,
- during excursions with handicapped or old age people

... this means, also in locations/situations where the use of bedpan or urine bottle is difficult or not possible due to the difficulties in keeping, emptying and cleaning an open container.

Environment and P-BAG

P-BAG (funnel and bag) is produced from the environmental friendly plastic polyethylene (PE). This means that the P-BAG can be disposed of as normal garbage or burned. If the P-BAG is burned, it releases 65% H₂O (water steam) and 35 % CO₂ (carbon dioxide).

How to use the P-BAG

The funnel handle shall be on the upper side of the bag. Men uses the P-BAG as if it is a urine bottle.

Women shall place the funnel right below the urethra and apply a small pressure to ensure a good fit. The funnel handle shall point away from the body. To enable a free flow of the urine into the bag, it should be checked that the bag is not folded.

After completion the funnel shall be held upright to enable the last droplets to flow into the bag.

Until next usage, the P-BAG can be hung on the funnel handle or placed on the floor with the funnel pointing upwards.

Before emptying the bag and removing the funnel, it is advisable to pour some water through the funnel to clean it's inside as well as the outside of the one-way valve of the bag.

Thereafter the bag can be opened at the tear mark in the upper left-hand edge of the bag and emptied without coming in contact with the urine.

Why should you upgrade from bedpan and bottle to a P-BAG

- P-BAG is suitable for both women and men
- P-BAG has a one-way valve that stops urine as well as it's odour from leaving the container
- P-BAG is comfortable and easy to use and gives the patient a better integrity
- P-BAG improves the working conditions and environment for the personnel
- P-BAG can be used everywhere, there is no risk of spillage and no need to empty and clean a urine container.

and

- P-BAG reduces the urine handling cost in hospitals, old age homes, nursing homes as well as in homecare.

This is how you can order

P-BAG is available in different packages:

Temporary usage: for example in emergency wards and during transport of patients, where the patient is only using a P-BAG for a short period of time..

Continuous usage: when a patient is using the P-BAG over an extended period of time, the funnel can be used for several bags.

Order number	Content / Carton	Temporary/Continuous
10 0005	30 funnels + 30 bags	T
10 0006	10 funnels + 100 bags	C
10 0007	100 funnels + 100 bags	T

This product information is presented to you by: